



SARAMIS™ Run-it Box

Quick Start Guide

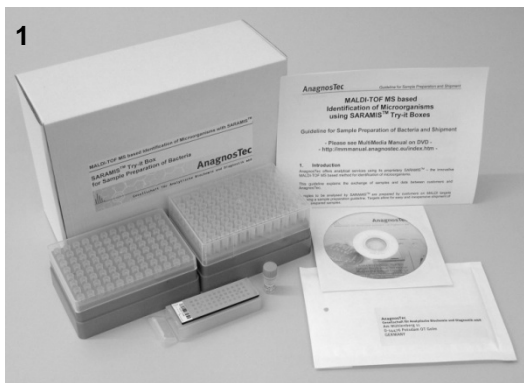

Content of your Try-it Box

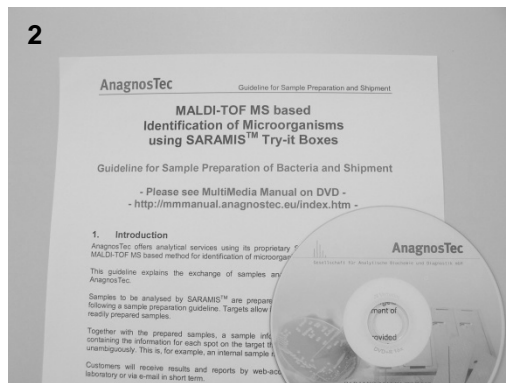


Read the sample preparation manual




3 Aerobic bacteria: 24 h
Anaerobic bacteria: 48 h


Pick-up a small amount of cell material

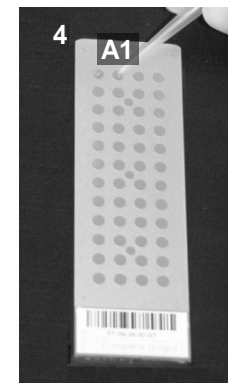

Apply cells except G3 & G4

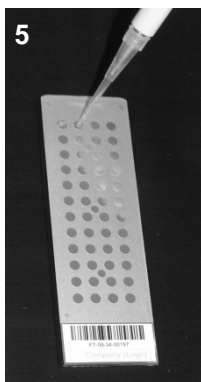

Add 1.0 µl matrix

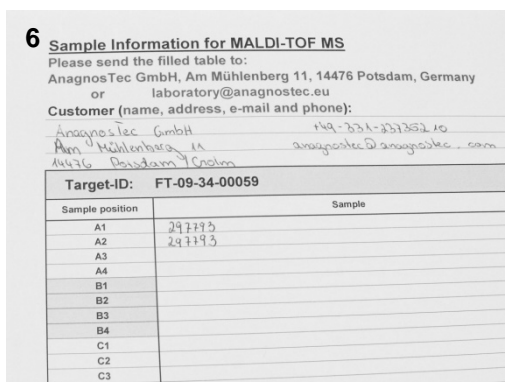

6 Sample Information for MALDI-TOF MS


Please send the filled table to:
AnagnosTec GmbH, Am Mühlenberg 11, 14476 Potsdam, Germany
or laboratory@anagnostec.eu

Customer (name, address, e-mail and phone):

Anagnostec GmbH 149-331-3335210
Am Mühlenberg 11 anagnostec@anagnostec.com
14476 Potsdam 76000

Target-ID:	FT-09-34-00059	
Sample position	Sample	
A1	291193	
A2	291193	
A3		
A4		
B1		
B2		
B3		
B4		
C1		
C2		
C3		

Complete the sample information form

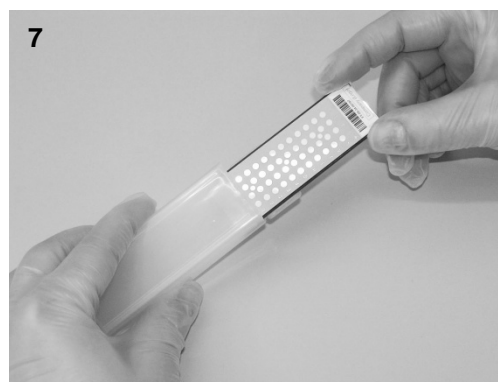

Store the prepared target in the mailer box



Return target and form to AnagnosTec



Caution: Target has to arrive Anagnostec 48 h after preparing the first position at the latest!!